

Tools Needed:

- Box Cutter
- #2 Philips head screwdriver

Uncrating the System

1. The device will arrive on a pallet, covered by a cardboard dust cover.
2. Remove any packing manifests or shipping documents for later use.
3. Cut the packing straps holding the dust cover.
4. Cut and remove the cardboard dust cover.



5. Remove the plastic cover and the upper foam.
6. Cut the packing straps and shrink wrap that secure the monitor to the stand (hold the monitor as you cut the shrink wrap to prevent it from falling)
7. **With the assistance of another person**, move the monitor box off the pallet and set it nearby with the wording on the box upright.



8. Remove the shrink wrap from the accessory box on the shelf and set the box aside.
9. Remove the shrink wrap from the wheelbase of the stand
10. **With the assistance of another person**, remove the stand from the pallet.



Installing the Monitor

11. The monitor will come fully assembled with mounting hardware and camera bracket.
12. Carefully cut the shrink wrap holding the camera and set the camera on top of the computer cabinet.





13. **With the assistance of another person**, tip the monitor box upwards to expose the tape joints on the bottom. Cut along the joints and set the monitor back down.



14. Lift the upper monitor box up off the base to expose the monitor



15. **With the assistance of another person**, lift the monitor up and onto the mounting bracket, setting the right-side monitor bracket between the two preset slide locks



16. Confirm that it is set between the slide locks. This ensures that the monitor is centered on the stand.



17. Set the stability brackets
- a. Unscrew the stability brackets from the back of the monitor.
 - b. Set them upside down, so that the stability bracket overlaps with the monitor mounting bracket and the letters are upright, then tighten the screws in their new position. Tuck under the wires.



18. Plug HDMI and touch cables into HDMI 1 and Touch 1 ports.



19. Plug the power cord into the power port and ensure that the switch is set to the ON position (Line).



Setting up peripherals

20. Open the accessory box containing the keyboard, air mouse, tv remote, and batteries.

21. Insert the appropriate batteries into each



22. Place them onto the VR shelf



Mounting and Adjusting the Camera

- 23. Set the camera on the mount and secure by tightening the attached thumb bolt up into the bottom of the camera.
- 24. Plug the unit into the wall, then turn on the computer and monitor. The system will launch into OmniVR automatically.



- 25. Choose any exercise and proceed to the patient placement screen.
- 26. Ensure that you are centered horizontally and vertically in the live view box in the upper left of the screen. If the patient is not centered, the camera may need to be adjusted until they are.

