

## 1903030 OmniSWD Packing Instructions



----- The Packaging Materials of the Diathermy box are shown above -----




1. Begin by inserting the base and casters in the two piece bottom foam set. You may need to lock the caster brakes and carefully tilt the unit to do this.



2. Guide the Casters into the slots.



3. Repeat for the opposite side.

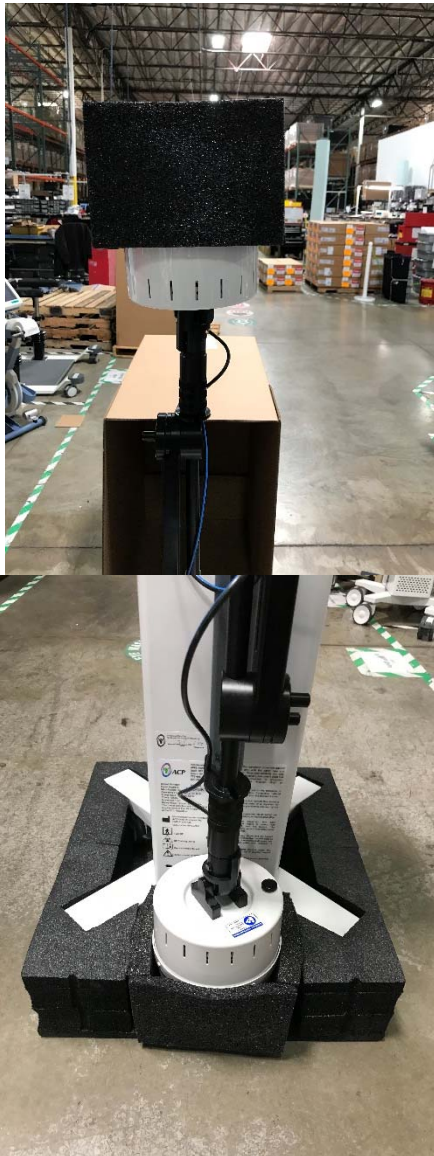

4. Place SWD Head Foam Piece on the top of the Applicator Head, then fold entire arm down to fit the head into the slot in the base foam set. You may have to extend the upper arm to get it into the slot.



5. Install the Top Foam Piece. The rectangle hole goes over the Arm Mounting Joint



6. Foam installed properly



7. Slide the unit into the box. Ensure the box Arrows are pointed upward.



8. Carefully tip the box with SWD until it is resting on its side.

8.5 Ensure power cable is wrapped, secured in sealed bag, and placed inside unit's box



9. Tape the box closed, ensuring that the seams are secure enough to shipping.



10. Lift box with Arrows pointing upward.  
Packaging is now complete.

For return instructions, see the yellow or green sheet affixed to the return box.